(EN)

SeptoPolisher

Silicone polishers for composites, glass ionomers and amalgams

SeptoPolisher is composed by abrasive silicone tips, indicated for finishing and polishing restorations. Available in three different shapes, it enables the work on all faces of any tooth. Depending on the strength applied it is possible to have both, finishing and polishing, in a single product.

- The more strength is applied the best finishing will be achieved in the restorative material.
 - The low strength provides the polishing, resulting in a smooth, uniform and bright surface.
- SeptoPlus Mandrel system that works by pressure, easy and fast to set, provides agility to switch between different shapes. Available in 3 different shapes: Cup (a), disc (b), point (c).

Use: Install the mandrel onto the contra-angle. Place the desired tip at the end of the mandrel. Apply the product dry, in low to medium rotation, with intermittent touches. The application with more strength provides the finishing of the restorative material. The application with low strength provides the polishing of the restorative material.

Warnings, precautions and restrictions: Professional use.If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. The use of tips and mandrels with similar products is of user's sole responsibility. Apply the product with rotation between 5,000 and 10,000 RPM. Use outside specified speeds may compromise performance (below 5,000RPM) and cause discomfort to the patient (over 10,000 RPM). Mandrel and tips must be sterilized in autoclave after each patient. Sterilization methods may cause a slight color changing on the tips, which does not affect its functionality.

Sterilization: Autoclavable (126°C, 147kPa, 16min). Before sterilization, proceed with the cleaning using water and enzymatic detergent.

Composition: Tips: Silicone rubber and aluminum oxide. Mandrel: Metal.

Storage: To be stored in a cool and dry place, protected from sunlight.

Connection with contra-angle: The SeptoPlus Mandrel is manufactured in compliance with specifications of ISO 1797.

Presentation:
010000010 – SeptoPolisher assortment refill – 8 units + 1 mandrel
010000011 – SeptoPolisher point refill – 8 units + 1 mandrel
010000012 – SeptoPolisher disc refill – 8 units + 1 mandrel
010000013 – SeptoPolisher cup refill – 8 units + 1 mandrel

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TDV Dental Ltd.