SeptoMatrix Cervical

Transparent matrix for Class V restorations

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Use: Select among the five available sizes the one which better adjusts to the tooth to be restored. Adjust the matrix to the instrument. If the cavity has a subgingival extension, promote the gingival retraction, insert the fast layer of composite, compress the matrix against the tooth forcing the composite to flow in the cervical region. Remove the excess from the edges of the matrix and polymerize for 10 seconds through the matrix. Remove the matrix and complete the polymerization according to the recommendations of the manufacturer of the composite. Perform the finishing and polishing procedures.

Warnings, precautions and restrictions: Professional use. The matrices are for single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operator. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Cleaning and sterilization: The instruments are autoclavable (126°C - 16min / 134°C - 18min). Before sterilization, proceed with the cleaning using water and enzymatic detergent. Cleaning: Carry out the cleaning with the aid water and enzymatic detergent 4 enzymes (Amilase, Protease, Lipase and Carboxylase) and with pure pH range between 6,5 - 7,5. Prepare the solution according to the manufacturer's instructions. The water should be potable. For a better effectiveness, it is recommended that the temperature be between 40° and 55°C. Submerge the products and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the pieces with absorbent paper or disposable wipes. Inspection: Check the product before each use, observing whether there is damage to the lips or cracks along the instrument, and do not use in such cases. Cracks are indicative of material fragility, which can cause the product to break.

Composition: Instrument: Polysulfone, Matrices: Polypropylene.
Storage: To be stored in a cool and dry place, protected from sunlight.
0105040016 - SeptoMatrix Cervical - 2 instruments + 10 matrices

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